

# VS-153-ADA

## Installation

1. Hold the Grab/Slide bar against the wall in the chosen location. Using a pencil, mark the drill hole positions (4 in total) for the lower and upper mounting brackets. If using as a Grab bar, be sure to fasten securely to a wall stud. (Note: upper mounting bracket height is adjustable)
2. To mount the brackets, use a No. 10 (0.1935 inches/5.5 mm) masonry drill bit to make a hole approximately 1 3/8 inches (35 mm) deep. Push the plastic anchors (4 in total) into the drilled holes, unless your mounting is to wall studs, where anchors can be omitted. (Note: plastic anchors provided may not be suitable for all types of wall construction, seek professional help if in doubt)
3. Fix upper mounting bracket and then the lower mounting bracket to the wall using the four (4) screws provided.
4. Push the bracket covers over the mounting brackets while paying close attention to cover orientation for proper placement.

Clean Grab/Slide bar regularly using a non-abrasive liquid bathroom cleaner.

